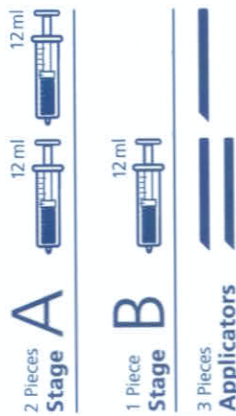
Guidelines to use Gynotal® Obstetric Gel (to be used by midwives and doctors only)

Dosing and Application

On 1st vaginal examination – introduce 3mls of gel using elastic applicator provided.

At every subsequent examination, apply an additional 3mls of obstetric gel.

If vaginal canal is dry, add sufficient fluid with gel. Sodium chloride or any aseptic solution can be used.

Add more gel after 15 to 30 minutes after amniotic sac ruptures.

Stage A gel – 1st stage of labor - dilatation phase

Stage B gel – 2nd stage of labor – expulsion phase

Obstetric gel should be used during the entire duration of labor. Average required dose is between 10mls to 30mls per birth.

Precautions:

It is sterile as long as the blister packing is not opened. Do not use of blister packing is damaged or open or expiry date has passed. After opening, the injectors must be treated as per rules for using sterile materials.

When the baby appears, the face (including eyes and nose) should be wiped with a dry cloth to prevent sliding when the baby comes out.

Biocompatibility and use in pregnancy : Has been tested in accordance with international guidelines and there are no indications of a harmful effect for the mother or new born.

Side effects: No side effect has been reported so far

Interaction: No cases have been reported to date.

Contraindications: Gynotal® Obstetric Gel must not be used in women who have allergic history against any of the ingredient of this product.

Importer and distributor :

Pengimport dan Pengedar :

INIQUARA PHARMA
(LLP0001002-LGN)

For More Information and Purchase
Please Contact :

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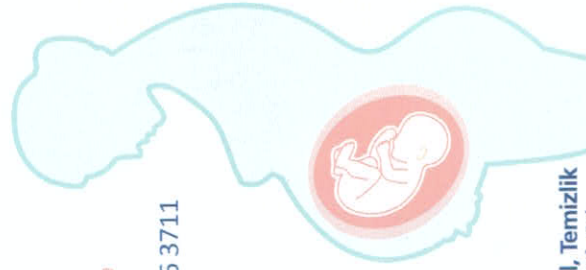
GYNOTAL

Protecting Mother & Baby

For Safer, Shorter and Smoother Birth process....

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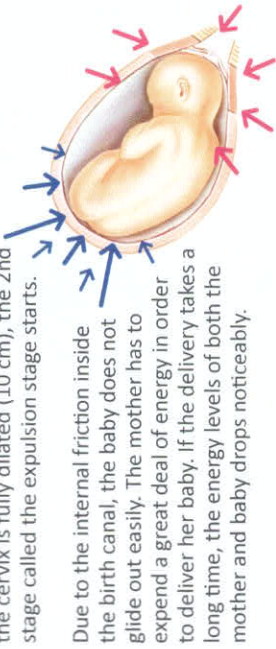
The Birth Process

The process of childbirth is divided into two stages. During the 1st stage, called the dilatation stage, the uterine cervix opens. When the cervix is fully dilated (10 cm), the 2nd stage called the expulsion stage starts.


Due to the internal friction inside the birth canal, the baby does not glide out easily. The mother has to expend a great deal of energy in order to deliver her baby. If the delivery takes a long time, the energy levels of both the mother and baby drops noticeably.

During the expulsion stage, the vaginal tissue or the perineum may tear spontaneously. This occurs in 45% of all births. The obstetrician will try to prevent uncontrolled tearing by making a cut in a safe direction. However, it is not always possible to prevent tearing.

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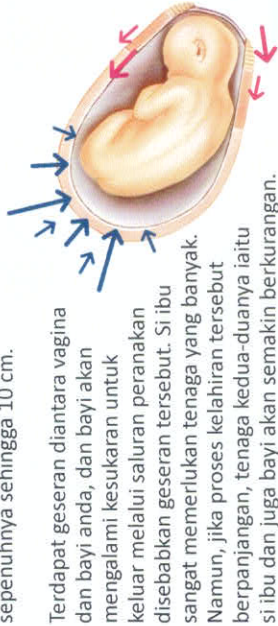What are the benefits of using Gynotal Obstetric Gel?

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**SHORTER
SINGKAT**

**SMOOTHER
SELESA**

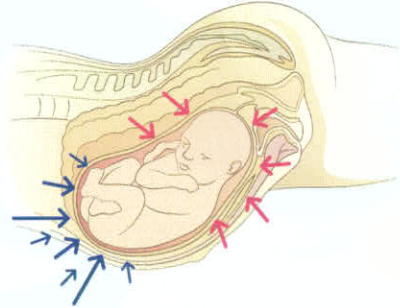
It is a specially developed lubricant to ease natural birth process. It is highly bioadhesive and is absorbed and forms a slippery surface inside the birth canal. It neutralizes the internal vaginal friction and facilitates the baby to glide out smoothly. It is sterile, non-allergenic and does not irritate the eyes or the skin of the baby and harmless even if the baby ingests it.

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The mother experiences better confinement. She can walk and sit more comfortably and it reduces uro-gynaecological problems.

It aids and eases deliveries for which a vacuum pump or forceps are used.

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